

HV2000 Replacement parts instructions


New 03-19-2008

Approved: Gene Kulon

Whenever a hand control, control box or transformer needs to be replaced use the following instructions.

Remove the old parts.

Hand control: un-staple the bottom of the pocket to allow the hand control to be slipped out of the pocket. Disconnect the hand control from the control box and remove the cable clamps holding the wire to the chair. Slide the wire out of the pocket leave the pocket opened for the new hand control.

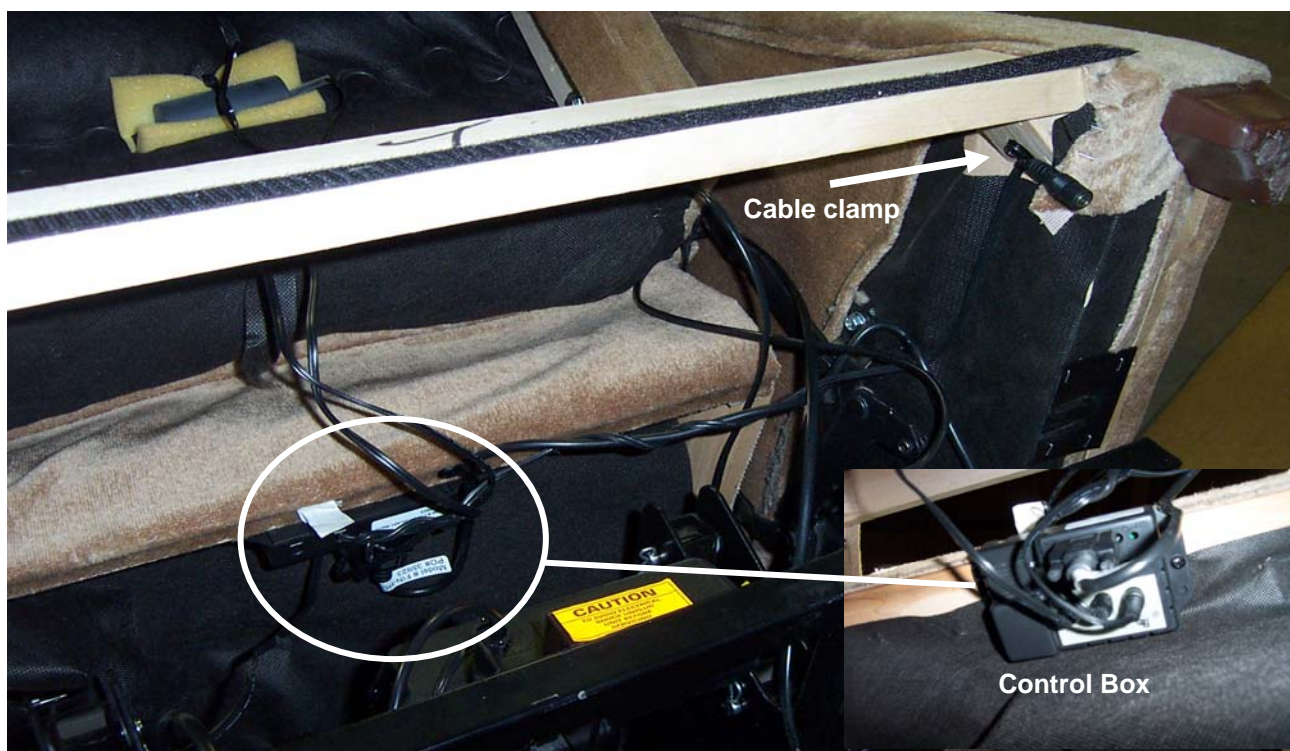


Control box: disconnect all the wires connected to the control box and unscrew it from the back of the seat. Unscrew the cable clamp holding the input power jack to the transformer. **The hand control, control box and transformer will not be reused. Please discard, do not return.**

HV2000 Replacement parts instructions

New 03-19-2008

Junction Block:

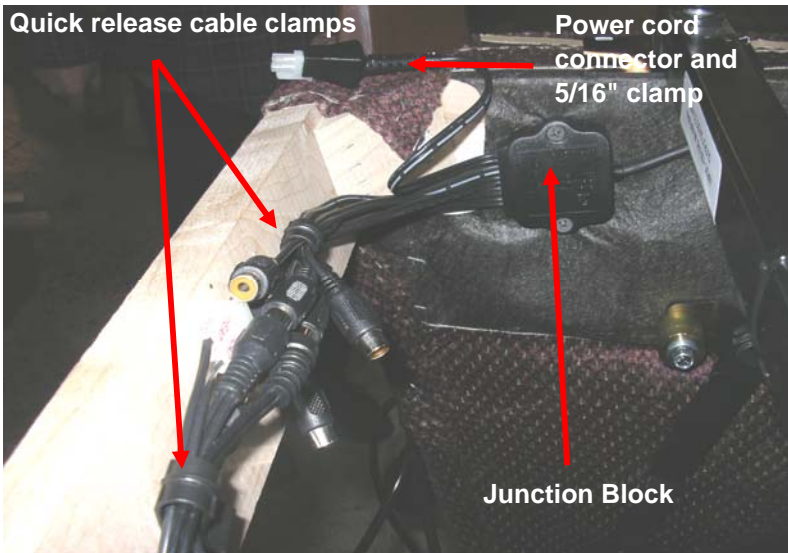


Screw the junction block to the lower right inside rear corner of the arm using two 1 1/2" screws provided. Plug in the vibrator wire and the heating pad wire. Screw the two quick release cable clamps onto the arm cross rail, using the 3/4" wood screws provided, and secure the wires. Place the power cord connector into a 5/16" cable clamp and screw it to the bottom of the arm along the right rear landing leg using a 3/4" screw provided.



The junction block is marked with zone numbers where the wires enter the block. Zone 1- upper back vibrator. Zone 2- lower back vibrator. Zone 3- seat vibrator. The vibrators are connected with speaker jack connectors and the heating pad uses a white push in connector. All connectors are specific to the item being connected and can not be plugged into the wrong connector.

Hand Control:

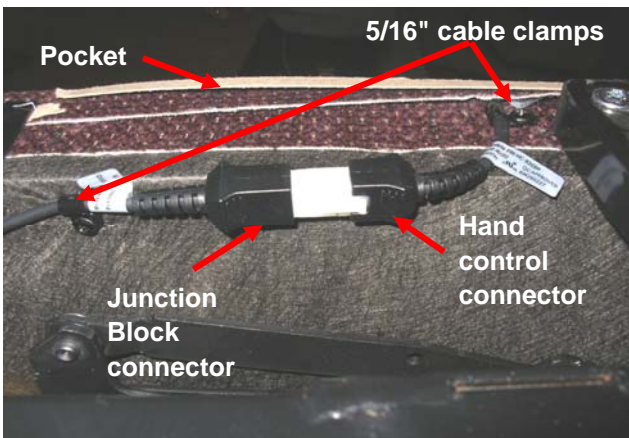

Slide the connector end of the hand control thru the pocket, from the inside, at the bottom of the arm. Plug the connector end of the hand control into the white connector coming from the junction block. Secure the wire on each side of the connectors using two 5/16" cable clamps and 3/4" screws provided. Close the bottom of the pocket by stapling it closed along its length being careful not to put a staple thru the hand control wire.

HV2000 Replacement parts instructions

New 03-19-2008



Secure the hand control into the quick release connector, attached to the side of the right arm, above the pocket. Read the operating instructions, on the care and use of your new heater vibrator system, provided with your replacement parts kit.