

A ZONE display

LED shows which zone is active.

B TIMER display

LED shows amount of time selected.

C HEAT display

LED shows which heat level selected.

D POWER key

Press to turn on massage—timer sets for 20 min. Press and hold key for 2 sec. to set timer for 30 min.

E ZONE keys

Press to turn zones (areas of body being massaged) on or off.

F MASSAGE key

Press key to toggle between WAVE, PULSE, or STEADY massage types.

G HEAT key

Press once for high heat, press again for low heat, and press once more to turn heat off.

H INTENSITY keys

Press "-" to decrease, and press "+" to increase intensity.


I SPEED keys

Press "-" to decrease speed, and press "+" to increase speed.

J SENSATE keys

Press "-" to decrease the massage wave overlap, and press "+" to increase overlap.

Tranquil-Ease® Model HC-3703H

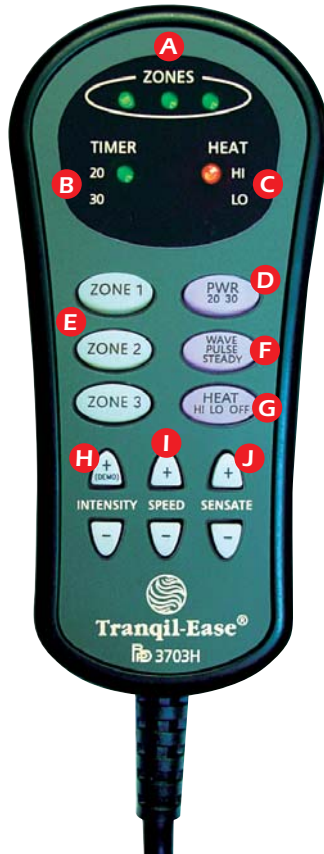

Raffel Product Development Co., Inc.

150 Progress Drive ■ Saukville, WI 53080

1-800-307-1058 ■ Local: (262) 284-6486

Fax: (262) 284-6535 ■ www.raffel.com



HC-3703H

User Guide

Copyright ©2004 Raffel Product Development Co., Inc.

Tranquil-Ease®




IMPORTANT SAFETY INSTRUCTIONS

WARNING: To reduce risk of burns, fire, electric shock, or injury to persons:

- READ INSTRUCTIONS BEFORE USING
- BURNS WILL RESULT FROM IMPROPER USE
- NEVER REMOVE COVER

- **IMPORTANT:** Patients using a pacemaker should consult a physician prior to use.
- This appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts.
- Close supervision is necessary when this unit is used by, on, or near children, invalids, or disabled persons.
- Use this unit only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer. Keep the cord away from heated surfaces. Do not use outdoors.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- Temperatures sufficiently high to cause burns may occur regardless of the control setting. Do not use on an infant or on a sleeping or unconscious person. Do not use on insensitive skin or on a person with poor blood circulation. Check the skin in contact with the heated area of the appliance.
- Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a service center for examination and repair.
- To disconnect, turn all controls to the off position, then remove plug from outlet.

WIRING DIAGRAM

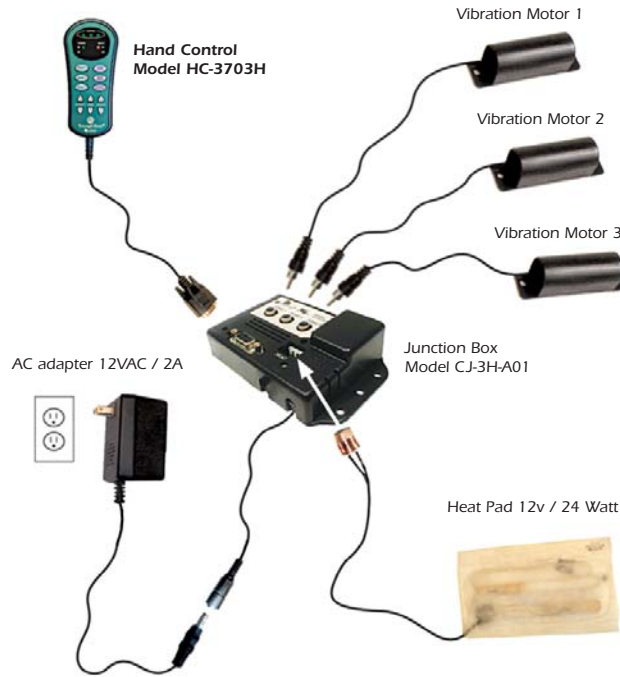

DANGER—TO REDUCE RISK OF ELECTRIC SHOCK:

- **Always unplug the unit from the electrical outlet immediately after using.**
- **Do not reach for an appliance that has fallen into water. Unplug immediately.**
- **Never use pins or other metallic fasteners with this appliance.**
- **Carefully examine the covering before each use. Discard the appliance if the covering shows any sign of deterioration, such as checking, blistering, or cracking.**
- **Keep Dry - Do not operate in a wet or moist condition.**

Symptoms	Possible Causes/Solutions
NO POWER	<ul style="list-style-type: none"> • Microprocessor Malfunction: Unplug unit, wait 30 seconds, then plug back in. • Unit plugged into switched outlet or bad outlet: Verify outlet is working and that switch is on if it is a switched outlet.
MOTORS/HEAT NOT WORKING	<ul style="list-style-type: none"> • Microprocessor Malfunction: Unplug unit, wait 30 seconds, then plug back in. • Loose/unconnected wires: Verify all motor/heat pad connections.

Operational Notes:

- Use only the wall transformer furnished.
- The memory will be lost when unit is unplugged.
- If system is accidentally unplugged while running, leave unplugged for 30 seconds before plugging in again.

SAVE THESE INSTRUCTIONS