

## Golden Technology Bed Assembly Instruction Form

PLEASE READ BOTH SHEETS BEFORE DELIVERY ATTEMPT

### Tools


This is for the Assembly of a Golden Tech. Bed. Tools will be required to complete the assembly. Here is a list of some of the tools required.

- Box Cutter – to open box and remove any packaging material
  - 3/8" Socket – mattress guard
  - 1/2" Socket – bed rails
  - 7/16" Socket – headboard brackets & swing away hinges
- \*Full Socket Set, Screwdriver, Hammer, and Adjustable Wrench are recommended in addition to the above tools.

### Bed Assembly

(See below diagram for references)

1. Remove the plastic bags containing the hardware, power cord, hand control and casters. Dispose of bags properly since they can be a hazard to small children.
2. Install the four casters as illustrated.
3. **Tools:** 3/8" socket. Lift the bed from the carton, position the bed on its side and install the mattress guard.
4. Position the bed with casters down, turning the bed to its upright position.
5. **Tools:** 7/16" socket. Plug in power cord and raise headboard. Unscrew the headboard brackets, turn them around, and then screw them back in. Note: It is not your responsibility to attach the bed to the customer's headboard. We would prefer if you didn't.
6. Place the mattress in position. Note: the removal and disposal of the plastic cover is necessary to insure proper ventilation and to avoid the possibility of mildew. Discard the plastic cover since it is a hazard to small children.
7. Insert the power cord plug into the wall outlet. Note: if the wall outlet does not have a three-prong receptacle (grounded type) an adapter must be used.

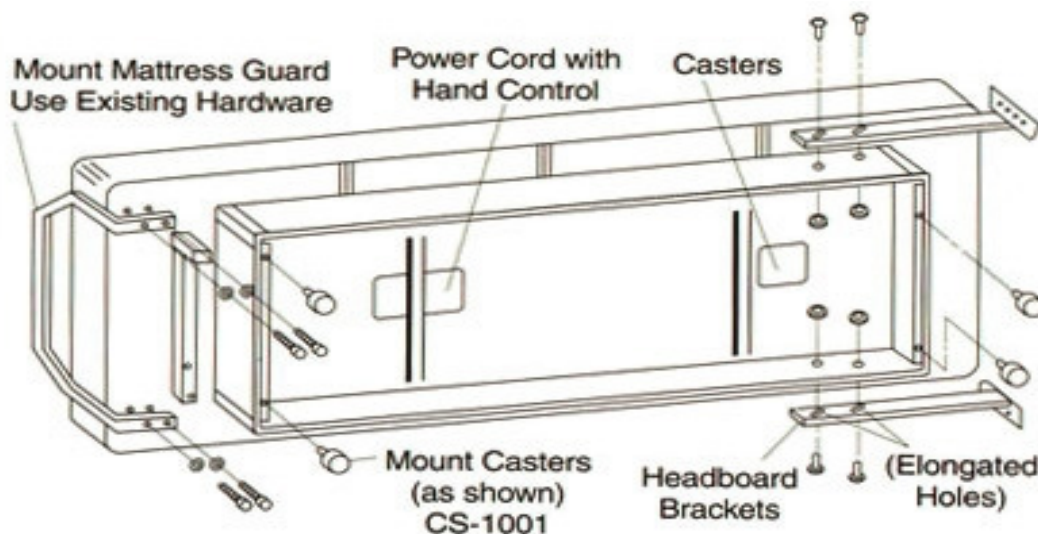

*See page 2 for more information*

## **King Size Bed**

The Golden Tech King size bed consists of 2 Twin beds. The two beds can be linked together to form the King Size bed. To form the King, have the Twins in close position as one bed. Raise the footboard section of each bed and place bed lock (which looks like a small crowbar) across each base from metal to metal. Lower the foot section and place mattress on bed bases.

## **Swing Away Hinges**

**Tools:** 7/16" socket wrench.

-The Swing Away Hinges are only included with the King Size Bed. They attach to the Headboard Brackets to attach to the headboard. You are not responsible to attach the Swing Away Hinges to the actual headboard.

## **Bed Rails**

**Tools:** 1/2" socket wrench.

Bed Rails come as half-length rails. They are not included with all beds. Use the 4 bolts included with the kit and 1/2" socket to install the Bed Rails.

## **Helpful Hints**

- This is a White Glove delivery. It requires a minimum of 2 men.
- If you feel it would be easier to un-box the bed outside, please be very careful and make sure to have the customer sign the release on the POD.
- A generic hardware bag is included with all beds. Not all of the hardware will be used depending on the model bed that the customer ordered. Please explain this to the consignee if they question why there may be left over pieces.
- One important thing to remember is to make sure that the bed is facing the correct way. Note that there are specific sides for the head and the feet. Make sure to position them the way the consignee requests.

## **What Not to Do**

- You are not required to remove an existing bed from the room or home. The consignee should have an area clear where they want you to set up the bed.
- Please do not attempt to attach the headboard brackets to the headboard.
- You are not required to explain to the consignee how the bed operates and functions. We only require that the bed is plugged in and powers on.

If you have any questions, please feel free to contact Golden Technologies at 800-624-6374 ext. 445.